

SEMEN PACKAGING WITH PERISTALTIC PUMP



Compact
Fast
Economic

The New machine for semen packaging CF400 with peristaltic pump



Peristaltic head pump mod. 300



Electronic check of weld temperature

1. Completely made in stainless steel
2. Ready to package in 15 seconds
3. Peristaltic pump also available with timer (optional)
4. No wasted bags from start to finish production
5. Electronic check of weld temperature
6. The economic and rational choice for medium and small dimension Boar Stud
7. Available with timer to improve operator's comfort, speed and dosing accuracy.

The New machine for semen packaging CF400 with peristaltic pump

Simple and reasonably priced, the System CF400 Machine fills and seals SACCO bags.

- The SACCO bags move one by one in front of the filling station.
- The bag filling operation uses a peristaltic pump with a geared mini motor, designed to operate at best even in hardest conditions.
- Nothing but the hose or tube touches the fluid, eliminating the risk of the pump contaminating the fluid, or the fluid contaminating the pump.
- The semen flow is controlled by a timer.
- As one bag is filled, the one in front is sealed.
- This efficient system enables the operator to package the semen doses with a single work station, reducing the amount of semen handling. The unit is mainly in stainless steel for easy cleaning.
- The practical output is approx. 380 to 400 doses per hour, depending on both the operator and laboratory procedures.
- The CF400 machine is simple to install, easy to use, and quick to maintain, it is a hot new technology.
- The CF400 is able to fill also different bags (IMV, ..)



CF400

Code	Description
CF4001	Manual SACCO System CF400
WM300	Peristaltic head pump mod. 300
CF400SIL64	Silicon tube Ø 6,4 mm – 10 mt
CF400SIL80	Silicon tube Ø 8,0 mm – 10 mt
CF400TEMP	Timer Switch for Pump